

Digital SoundLink

DSL SK 50

BONE CONDUCTION HEARING AID EARPHONE

Hear Clearly. Live Comfortably.



Crystal Clear Sound



Comfortable & Stylish



Ultra-Long Battery



3 Listening Modes



Left / Right Volume



Safe & Non-invasive

| Better Speech Clarity

| Ideal for Daily Use

ABOUT BONE CONDUCTION TECHNOLOGY

Bone conduction technology transmits sound through gentle vibrations of the skull bones directly to the inner ear, bypassing the outer ear and eardrum. This helps users hear sound more clearly without placing a speaker deep inside the ear canal.



KEY FEATURES



Left / Right Channel Adjustment

Adjust volume separately for each side to support balanced hearing.



Ultra-Long Battery Life

Built-in 750mAh polymer battery provides approximately 20 hours of use.



Stylish, Lightweight Design

Comfortable for daily wear with a portable control box and clothing clip.



3 Channels Pickup Mode

Choose from 3 listening modes according to different hearing needs and environments.

WHO CAN BENEFIT?

Suitable for people who need everyday hearing support, especially those who prefer an open-ear wearing style or have difficulty using ordinary in-ear hearing aids. Hearing results may vary by individual hearing condition.

HOW TO USE

- 1 Insert the earphone cable into the control box.
- 2 Clip the control box onto your clothing using the back clip.
- 3 Place the bone conduction vibrators in front of the ears.
- 4 Adjust volume and mode according to your comfort.



OPERATION INSTRUCTIONS

- Press and hold the Power button for 3 seconds to turn the device on or off.
- Short press the Mode button to switch between 3 listening modes.
- Use the + and - buttons to increase or decrease volume.
- LED indicators show current mode, charging status and battery level.

SPECIFICATIONS

Hearing Aid Type	Bone Conduction
Battery	750mAh Polymer Battery
Battery Life	Approx. 20 hours
Charging Time	Approx. 2 hours
Speaker Type	Bone Conduction Speaker
Frequency Response	250Hz - 5500Hz
Sound Pressure Level	118dB
Microphone Sensitivity	-38 +/- 1dB
Charging Cable	Type-C, 30cm

OPERATION GUIDE

Function	Operation	Indicator / Prompt
Power On / Off	Press and hold power button for 3 seconds	Light on/off; voice prompt: Power On / Power Off
Mode Switching	Single click the mode button	Mode 1: DI, Mode 2: DI-DI, Mode 3: DI-DI-DI
Volume	Press + / -	Each press beeps. Max/min volume gives repeated prompt.
Low Battery	Automatic alert	Third indicator blinks; prompt: Low Battery
Charging	Use Type-C charging cable	Below 30% red, 30-70% yellow, above 70% green; full charge green stays on

MODES EXPLAINED

Mode 1 (DI): Low-frequency enhancement for users with comparatively more low-frequency hearing loss.

Mode 2 (DI-DI): Full-frequency balanced enhancement for more general hearing support.

Mode 3 (DI-DI-DI): High-frequency enhancement for users with comparatively more high-frequency hearing loss.

WARRANTY CARD

Customer Name: _____

Date of Purchase: _____

Contact Number: _____

Warranty Period: _____

Model: **DSL SK 50**

Dealer Stamp & Signature: _____

Remarks: _____

Please keep this warranty card for future warranty and service support.

SAFETY NOTES

Set a comfortable volume before use. Do not use at excessive volume for long periods. Keep the device dry and clean. Charging should be done with a suitable Type-C cable and safe power source. If discomfort, ear pain or sudden hearing change occurs, stop use and consult a hearing professional.

CONTACT



Head Office

Offakamp 22 - 15b, 22529 Hamburg, Germany



Email: info@digitalsoundlink.com



Website: www.digitalsoundlink.com